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The screenshot shows a patient information form with the following fields and annotations:

- Patient ID: E60003-14-07-22-1
- Last Name, First Name, Middle Name: [Redacted]
- DOB (mm/dd/yyyy): [Redacted]
- Age: [Redacted]
- Sex: female
- A red circle highlights a small icon in the top right corner of the form.
- A red arrow points from the text "phone/email entry" to the "female" text.
- A red box highlights the "DICOM field Indication" field.

Exam Date	Exam Time	Exam Type	M	Img	Application	Exam Size

The screenshot shows a patient information form with the following fields: Patient ID (1083351), Last Name (MUSTERFRAU), First Name (SUSI), Middle Name, DOB (05/01/1993), Age (21), Sex (female), and a green checkmark icon labeled '1'. Below the form are tabs for GYN, ABD, CARD, TR, VAS, SM P, PED, CEPH, and MSK. A pop-up window titled 'Patient's Email address and Phone number' is open, showing Patient Name (MUSTERFRAU SUSI), Email Address (susi.musterfrau@gmail.com), Phone Number (004913800000) labeled '2', and Send image via (Email & MMS). The pop-up also has sections for Email Options (Email send mode: anonymized, Send as: JPG, Size: Original, Quality: High) and MMS Options (MMS send mode: anonymized). The pop-up has 'Cancel' and 'OK' buttons, with the 'OK' button labeled '3'.

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The screenshot shows a search criteria form with the following fields: Search key (Patient ID), Patient ID, Accession #, Procedure ID, Start Date (05/17/2011), Station Name, End Date (05/17/2011), Station AE Title, and a Search button. Below the form is a table titled 'Scheduled Exams' with the following columns: Date/Time, Patient Name, Patient ID, Accession #, Exam Type. The table contains one row: 16/05/2010, 12:58, NAME F L, newID, AN, RPO. At the bottom of the form, there are buttons for 'Exit', 'Select', and 'Start Exam', and a status bar showing 'Items found: 1' and 'Last update: 05/17/2011, 10:46'.

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